



# VeroNobis™ Oral Retractor Instructions

The VeroNobis Oral Retractor™ with its unique patented design is a totally new dental instrument with many key applications.



**Figure 1:** VeroNobis Oral Retractor™ in place with cotton roll firmly tucked under the lingual retractor and the entire Parotid duct opening covered by the buccal retractor with dry angle. This provides for a completely retracted quadrant, floor-of-the-mouth protection and total salivary control. *Please note that the VeroNobis™ retractor is channeling the saliva distally for easy evacuation and ensures a dry operating field.*

## Recommended lingual technique:

1. Ask patient to place and hold the VeroNobis™ on their tongue as one would with a lollipop.
  2. Now that familiarity has been established, reposition the VeroNobis™ as needed, keeping it flat and slightly angled as in Figure 1.
- Performing steps 1 and 2 will ensure patient comfort, enhanced tongue relaxation, minimized gag response and salivary control.

## Placing pit & fissure sealants



- Buccal and lingual retraction and isolation for placement of etch
- Will easily accommodate both the retractor and curing light in smaller mouths
- Prevents the raising of the floor of the mouth and expression of tongue over the surface of the setting sealant material
- Etch can be washed off without removing retractor

## Laser procedures



- Excellent isolation and retractor for laser, electrosurgical and periodontal procedures
- Contoured comfort
- Optimal sight lines
- Non-reflective, non-conductive
- Single-use retractors available in convenient 25-count bags